



ASSEMBLY INSTRUCTIONS

DELUXE MOBILE LEAD SHIELD ON T-BASE



COPYRIGHT ©2025. ALL RIGHTS RESERVED

PHILLIPS-SAFETY.COM

ABOUT PHILLIPS SAFETY





Phillips Safety is the industry leader in occupational safety eyewear. In business for over 100 years, you are being served with the dedication and craftsmanship of a family owned company.

Phillips Safety has been producing occupational safety glasses since 1905 and continues to develop and produce the highest quality glasses and lenses today. A leading manufacturer of specialty safety eyewear in the United States, Phillips Safety adheres to strict ANSI Z87-2+ and EN 207 CE standard certifications, holding itself to the highest

standard and prides itself on producing glasses with the finest quality materials available.

All safety glasses produced and sold by Phillips Safety meet strict industry standards for military, medical, research laboratories, and a variety of other industries.

With Phillips Safety's dedicated team, you can rest assured knowing that all your prescription safety needs are met.

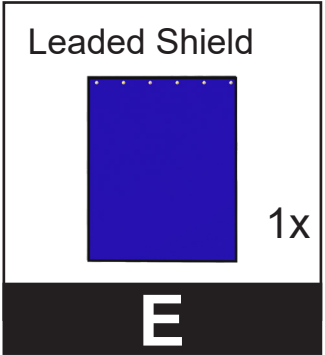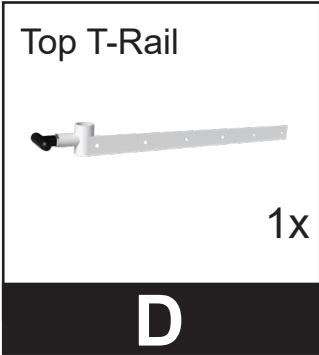
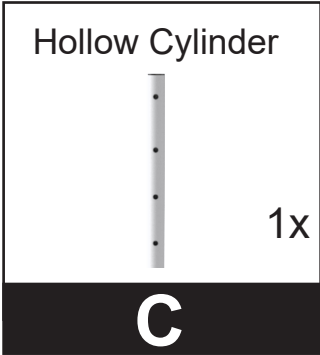
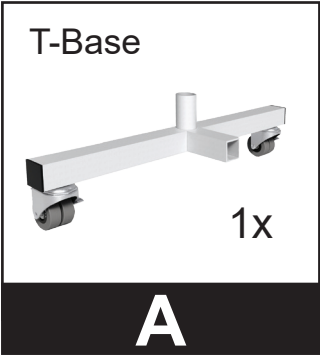


ASSEMBLY INSTRUCTIONS
MOBILE LEAD PORTA SHIELD

STEP 1


1. Get the T-Base **A** and the Straight One Wheel Base **B**.
2. Attach the T-Base **A** to Straight One Wheel Base **B**. The beam will go over the smaller square rod.
3. Tighten Allen Set Screws into place.

PARTS NEEDED

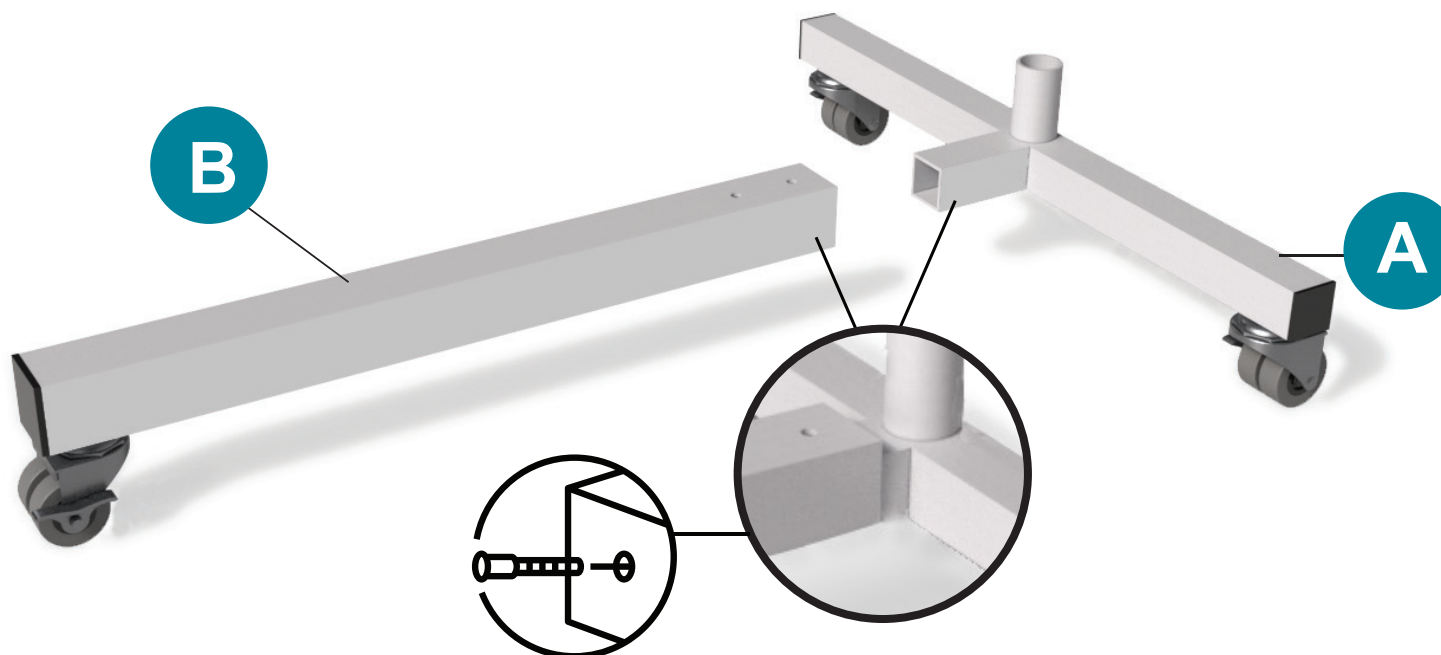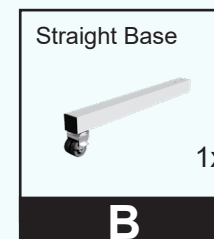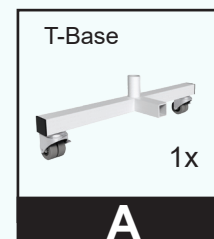


T-BASE ASSEMBLY

4. Insert the Hollow Cylinder **C** into the finished T-Base **A + B**.
5. Make sure the Hollow Cylinder **C** is straight and the holes are facing outward.
6. Secure into place.

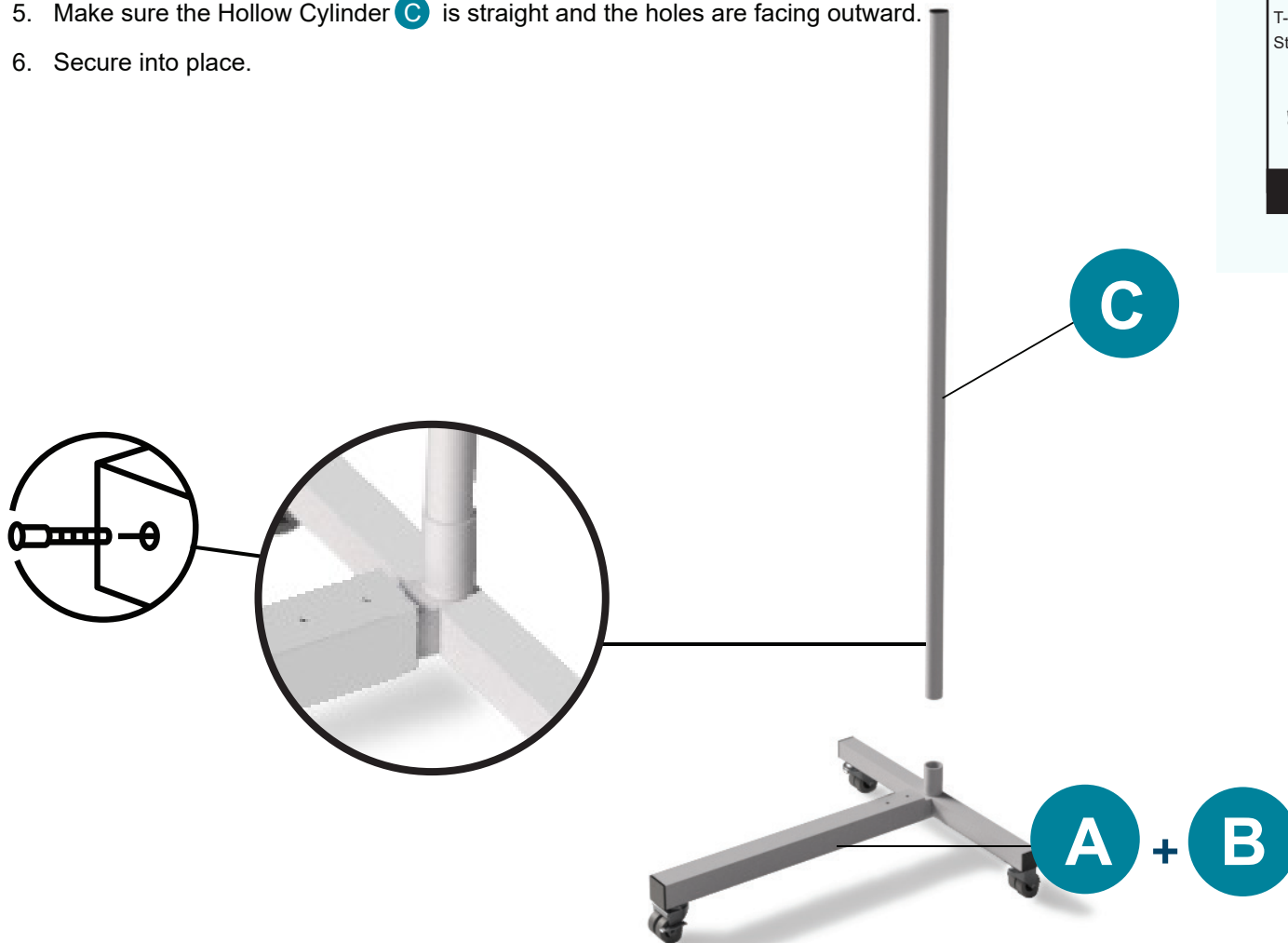


PARTS NEEDED

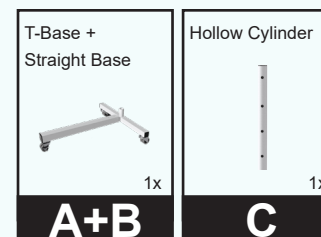


ASSEMBLY INSTRUCTIONS
MOBILE LEAD PORTA SHIELD

STEP 3

7. Attach the Top T-Rail **D** into the assembled Hollow Cylinder.
8. Pull the spring loaded black handle outwards.
9. Align the hole of the Top T-Rail **D** with the Hollow Cylinder **C**.

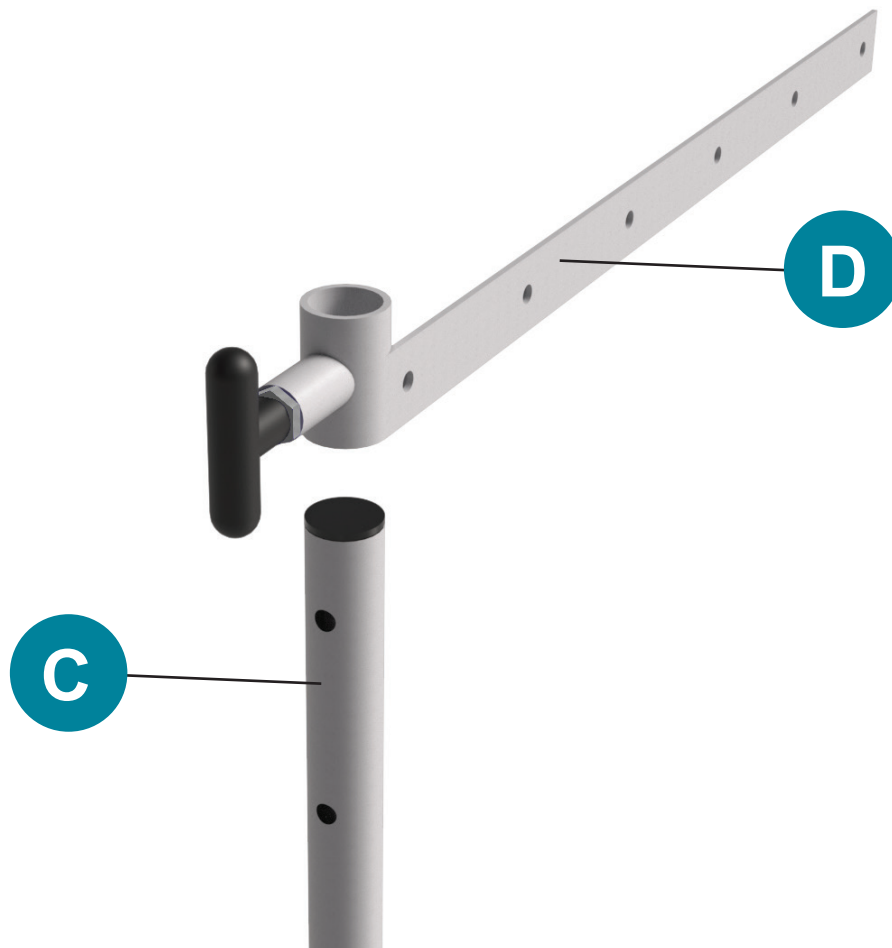

PARTS NEEDED

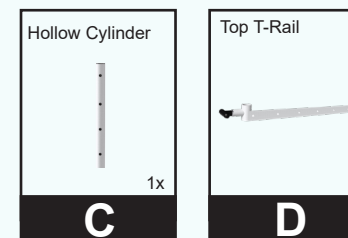

TOP TRAIL

10. Align the Lead Shield **E** with the Top T-Rail **D** holes.
11. To secure the Lead Shield **E** in place use the hardware in the following order:


G + **H** + **E** + **D** + **H** + **F**




PARTS NEEDED

Top T-Rail  1x D	Leaded Shield  1x E	
1/4 Wing Nuts  6x F	3/4 Phillips Head Bolts  6x G	1/4 Steel Washers  12x H

FRONT

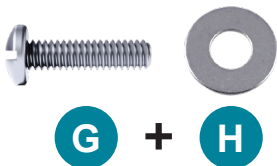


BACK

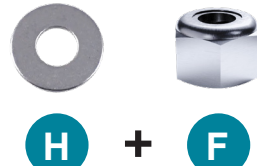

LEADED SHIELD


12. Pull plunge pin handle out.
13. Slide arm up/down to desired height.

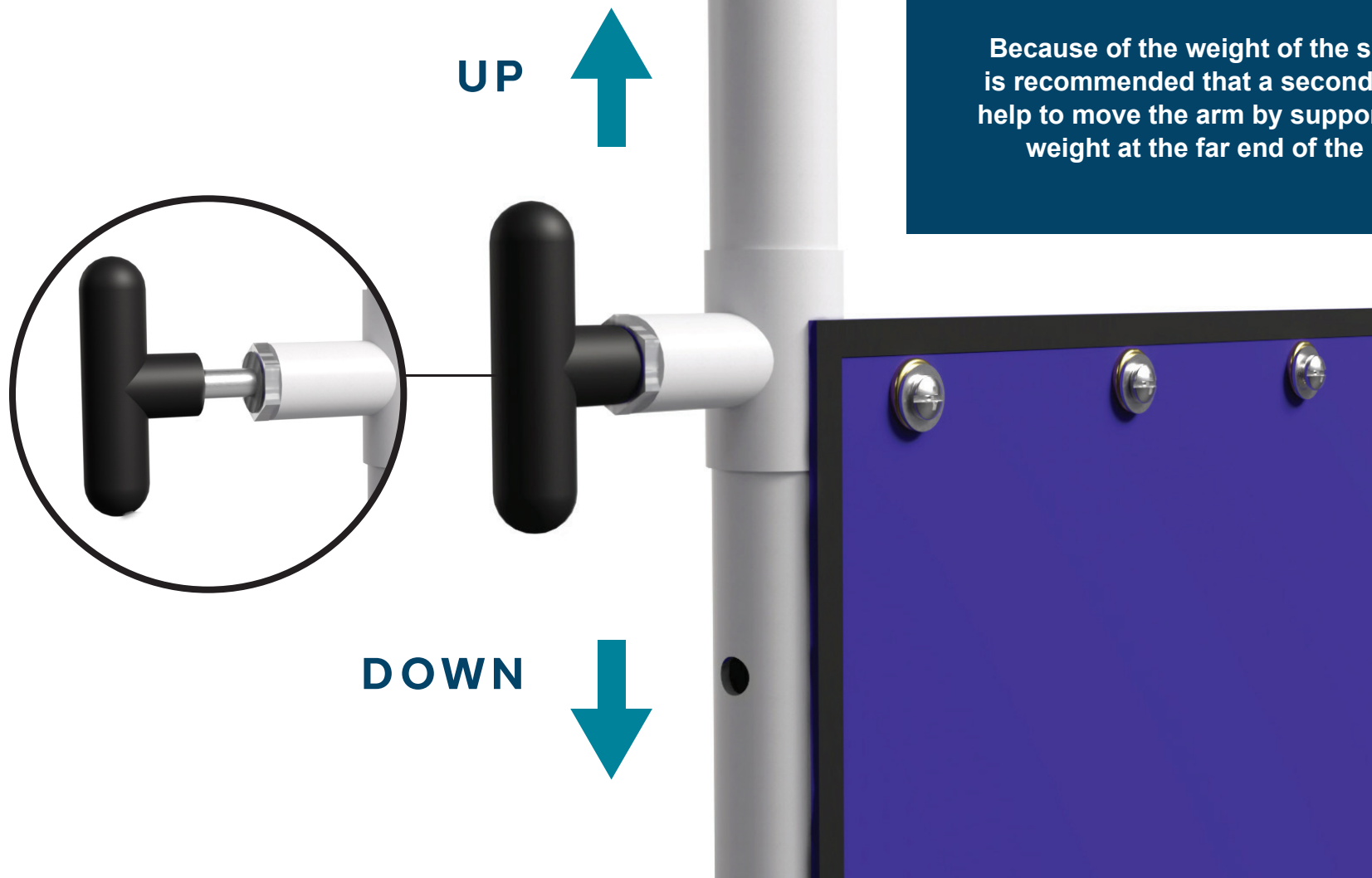

ASSEMBLY INSTRUCTIONS
MOBILE LEAD PORTA SHIELD

STEP 6

14. Tighten all loose screws and bolts.

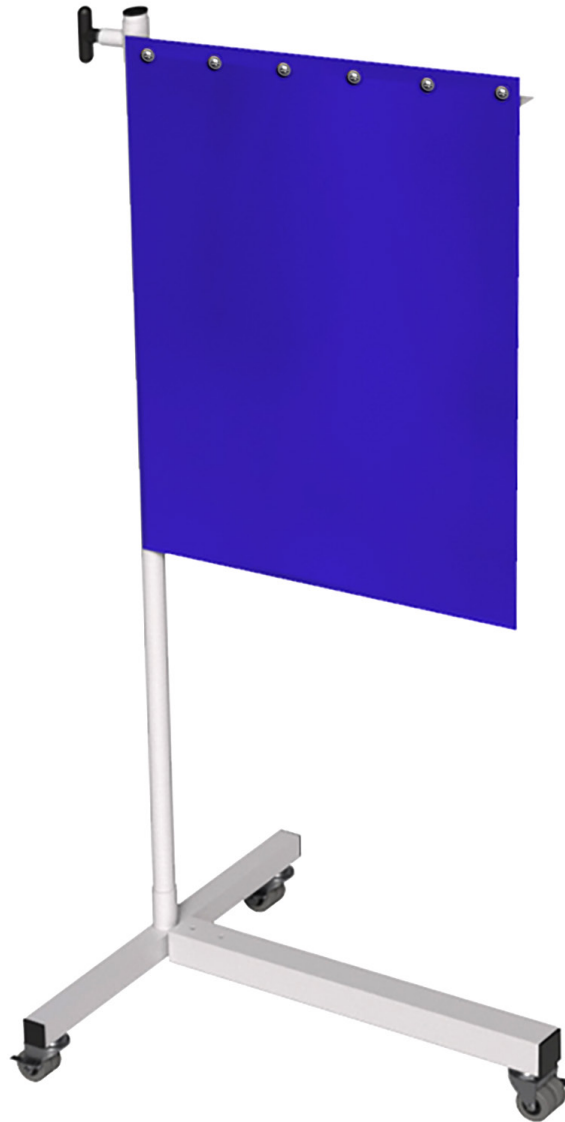

FINAL PRODUCT

CONTACT

Should you need any further information,
please do not hesitate to contact us.

123 Lincoln Blvd, Middlesex, NJ 08845, USA 

+1 (866) 757 1307 

service@phillips-safety.com 

www.phillips-safety.com 