

CLEANING & CARE GUIDELINES

Arm Boards

INSTRUCTIONS OF USE

PHILLIPS-SAFETY.COM

COPYRIGHT ©2025. ALL RIGHTS RESERVED



PHILLIPS SAFETY
PRODUCTS INC.

Arm Boards protect a patient's arms/legs during interventional procedures while securing them alongside the Operating table. The Arm Boards are designed to reduce the chance of having the arm pinched or crushed as well as eliminating arm drops during surgical procedures.

USE

Slide the toboggan under the Operating table pad and weight of the patient will hold the arm board in place.

Contoured top offers stabilizing support for clinician's arms while shielding against sharp edges during procedures.

Protects the patient and the lines connected during procedures

ATTENTION

Toboggan Arm Boards should be cleaned and disinfected after each use.

CLEANING

Wash arm board with a mild detergent and water. Do not use abrasive cleaners or scouring pad because this will scratch the arm board.

DISINFECTING

Use only disinfectants containing 70% Isopropyl alcohol (IPA) or 3-5% Hydrogen peroxide.

70% concentration of IPA is CDC approved for Covid-19 disinfection.

Do not use any disinfectant that contains dimethyl or diethyl benzyl ammonium chloride. Any form of ammonia (or -amines), in any amount, will cause cracks in the arm boards in a short amount of time.

MAINTENANCE

Before each use, arm boards should be inspected for damage (i.e. cracks).

If cracks are noted, the arm board should not be used.

CONTACT

Should you need any further information,
please do not hesitate to contact us.

123 Lincoln Blvd, Middlesex, NJ 08845, USA 

+1 (866) 757 1307 

service@phillips-safety.com 

www.phillips-safety.com 